

## VETRONIC ICON OXYGEN ANALYSER - CLEARANCE

VSL-ICON-CL



## PRODUCT DESCRIPTION

### ICON - In-Circuit Oxygen aNalyser

**ICON is an in-circuit oxygen analyser for use in veterinary operating theatres and intensive care wards. There are many situations where knowing the level or percentage of oxygen is beneficial, but monitoring or measurement is difficult.**

The ICON unit can be placed on the fresh gas outlet of an anaesthetic machine or within the re-breathing limbs of a circle system, wherever the oxygen concentration needs to be known.

This is particularly useful where breathing systems are not run with 100 per cent oxygen and different oxygen levels are required. It is also useful to know the actual concentration of delivered oxygen from an oxygen concentrator, or oxygen generator, as this may vary with load and flow rates.

It does not require a pump or delivery system to deliver gas to its sensing element. The active element is very close to the surface and this enables accurate measurements to be made in open-air environments, such as oxygen tents or oxygen cages.

ICON can be supplied with magnetic or velcro mounts for easy attachment to cage sides. ICON has a good response time, stabilising to a reading within 10-15 seconds. It uses a standard single AA battery which has a lifespan of approximately one year when used for eight hours a day, every day.

The sensing element is a long-life galvanic sensor which has a working life of approximately two to five years, based on an oxygen exposure of 100 per cent, to 21 per cent, respectively.

#### Helpful Videos:

[Vetricon ICON Oxygen Analyser - Demonstration](#)

#### Key Features:

- Early detection of hypoxic gas mixtures
- Easy-to-read screen
- Simple to use

- Reliable and low cost
- Small, unobtrusive unit
- Runs on one standard AA battery for a year
- Galvanic cell has a two to five-year lifespan
- Simple re-calibration procedure
- Swivel main body for preferred viewing position
- Fitted with standard 22mm and 15mm connectors
- Auto switch-off after eight hours continuous use
- Designed and manufactured in the UK

**The ICON comes supplied with everything you need to use the unit out of the box, including:**

- 1 x Operating Manual
- 1 x VS0015 x2 magnetic strips
- 1 x VS0016 x2 Velcro strips

## ADDITIONAL INFORMATION

<b>Animal</b>	N/A
<b>SKU</b>	VSL-ICON-CL

### Good to know...

At Burtons we know you have a choice to shop with us, that is why we always aim to delight our customers with a fast, easy, shopping experience. Making and sourcing only the best products we can find to fit your needs.  
**Price Matching:** We regularly check all our prices against competitors so you don't have to. If you do find a like for like product cheaper we will aim to not just match it, but beat it!  
**Backorders:** All back ordered items will be ordered and shipped as quickly as we can to you. Special order items may take up to 4 weeks. Once ordered a member of our team will be in contact to keep you informed of estimated delivery dates.  
**Returns Policy:** You may return new, unworn or unused products within fourteen (14) days of delivery for a full refund of the cost of the goods, or an exchange if preferred. This is on top of your statutory rights